
Instructions for Use  
**Ambu® ResCue Mask™**




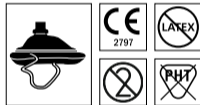
**Ambu A/S**  
Baltorpbakken 13  
DK-2750 Ballerup, Denmark  
T +45 72 25 20 00  
F +45 72 25 20 50  
[www.ambu.com](http://www.ambu.com)

US: Rx only

492.4201.40 - V07 - 2020/04 - Printed in China - Ambu A/S - TCC 10787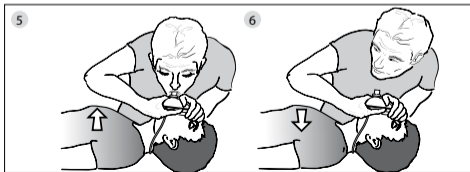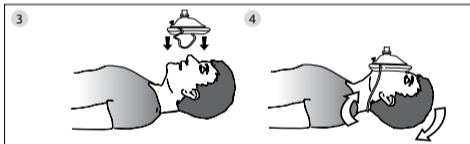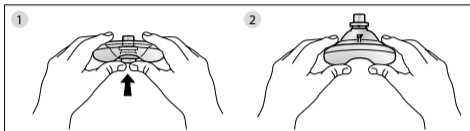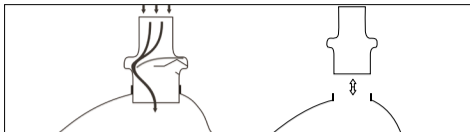

<b>EN</b>	This product is not made with natural rubber latex nor phthalates	<b>LT</b>	Gaminio sudėtyje nėra nei natūralaus latekso, nei ftalatų
<b>BG</b>	Този продукт не е произведен с естествен каучуков латекс или фталати	<b>LV</b>	Šis izstrādājums nav veidots no dabīgās gumijas lateksa un ftalātiem
<b>CS</b>	Tento výrobek není vyroben z přírodního latexu ani ftalátů	<b>NL</b>	Dit product is niet vervaardigd met natuurlijke rubberlatex of ftalaten
<b>DA</b>	Denne produkt er hverken fremstillet med naturgummilatex eller ftalater	<b>NO</b>	Denne produktet er ikke fremstilt av naturlig gummilatex, og inneholder ikke ftalater
<b>DE</b>	Dieses Produkt enthält weder Naturlatex noch Phthalate	<b>PL</b>	Ten produkt nie został wykonany z lateksu z kauczuku naturalnego ani z ftalanów.
<b>EL</b>	Αυτό το προϊόν δεν έχει κατασκευαστεί με φυσικό ελαστικό λάτεξ ή φθαλικές ενώσεις	<b>PT</b>	Este produto não é fabricado com borracha de látex natural nem ftalatos
<b>ES</b>	Este producto no está fabricado con látex, ni contiene ftalatos	<b>RO</b>	Acest produs nu este fabricat din latex din cauciuc natural sau din ftalați.
<b>ET</b>	Toode on valmistatud ilma loodusliku kummilateksi ja ftalaatideta	<b>RU</b>	В производстве данного продукта не используются фталаты и латекс из природного каучука
<b>FI</b>	Tämä tuote ei sisällä luonnonkumilateksia eikä ftalaatteja	<b>SK</b>	Tento výrobok nie je vyrobený z prírodného gumeného latexu ani ftalátov.
<b>FR</b>	Ce produit n'a été fabriqué ni avec du latex de caoutchouc naturel ni avec des phthalates	<b>SL</b>	Ta izdelek ni narejen iz lateksa naravnega kavčuka ali ftalatov
<b>HR</b>	Ovaj proizvod nije napravljen s lateksom od prirodne gume ni s ftalatima	<b>SV</b>	Produkten innehåller inte naturgummilatex eller ftalater
<b>HU</b>	A termék természetes latexgumi és ftalátok felhasználása nélkül készült	<b>TR</b>	Bu ürün doğa! kauçuk lateks veya ftalat kullanılarak imal edilmemiştir
<b>IT</b>	Il prodotto non è realizzato né con lattice di gomma naturale né con ftalati	<b>ZH</b>	本产品不含天然乳胶和邻苯二甲酸盐
<b>JA</b>	本製品に天然ゴムラテックスあるいはフタル酸エステルは使用されていません		

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## Contents

## Page

1. Use on adults and children.....	3
2. Use on infants.....	4
3. Use with oxygen (optional).....	4
4. Use with other ventilation equipment.....	5
5. Instruction for handling of product after use .....	5
6. Cleaning.....	5
7. Technical specifications.....	6
8. Material .....	6
9. Temperature limits.....	6

The Ambu® ResCue Mask™ enables performance of mouth to mask ventilation. Additionally it provides a physical barrier between the rescuer and the victim, eliminating direct contact of the rescuer's lips with the unknown subject.

Proper training in ventilation and CPR should be obtained before using this device. Practice using your ResCue Mask on training manikins so that you are ready for any emergency that might occur. In the event that you do not have your Ambu ResCue Mask start CPR without delay.

After removal of the valve, the Ambu ResCue Mask can be attached to a resuscitator if available.

### **Start CPR without delay**

#### **CAUTIONS**

- Ambu ResCue Mask is intended to be only used by trained users.
- Make sure that the valve is properly attached to the mask before starting ventilation. If not properly attached it may result in insufficient ventilation. If the one-way valve is replaced after the mask has been used, the replacement valve should only be inserted halfway into the mask and then be pressed fully into the mask just before starting the ventilation, in order to secure proper attachment.

### **1. Use on adults and children**

Carefully inspect all parts of damage or excessive wear and replace, if necessary.

- Remove the mask/valve assembly from case. Push out the soft dome with your fingers and ensure that the one-way valve is in place.
- Put on the protective gloves if available.
- Clear the airway of any liquid (blood, saliva and gastric content).
- Place the mask over the patient's mouth and nose. Position the elastic rubber band around the head if available.\* Apply the mask to the patient's face using the thumbs of both hands. Lift the jaw into the mask with the remaining fingers by exerting pressure behind the angles of the jaw (jaw thrust). At the same time, press the mask onto the face with the thumbs to make a tight seal.

- Blow through the inspiratory valve and watch the chest rise. Stop inflation and observe the chest falling. The seal between the face and mask can be reduced by adjusting the contact pressure, altering the position of the fingers and thumbs, or increasing the jaw thrust.
  - Continue rescue breathing until the patient responds or you are relieved by a higher medical authority.
- \* The Ambu ResCue mask comes both with and without an elastic rubber band.

#### **NOTE**

If the patient vomits; remove the ResCue Mask and clean both the patient's airway and the mask with your gloved fingers. If you should contaminate either the valve mouthpiece or any of your exposed skin surfaces, use the alcohol wipes to clean these areas immediately. Resume rescue breathing promptly.

### **2. Use on infants**

The Ambu ResCue Mask may be reversed to better fit an infants face. Place the nose section of the mask under the infants chin. Seal the ResCue Mask as described under the Adult/Child section and ventilate using the previously described technique.

The one-way valve of Ambu ResCue Mask can be connected to a face mask with 15mm standard male connector.

### **3. Use with oxygen (optional\*)**

The Ambu ResCue Mask may be used with supplemental oxygen. Attach the oxygen tubing to the optional oxygen inlet and set the flow rate. A head strap is included on all models that have the optional oxygen valve. If the patient is spontaneously breathing, use the head strap to position the ResCue Mask on the patient's face. If the patient is NOT breathing, begin mouth to mask ventilation immediately. The use of supplemental oxygen to the Ambu ResCue Mask will increase the delivered oxygen to the patient. The rescuer will not be affected by the use of supplemental oxygen.

\*ResCue Mask with Oxygen Inlet is Rx Only in the USA.

#### 4. Use with other ventilation equipment

The Ambu ResCue Mask may be used with Bag-Valve Mask ventilation devices. Simply detach the valve/filter assembly and attach the ResCue Mask to the device.

#### 5. Instruction for handling of product after use

The Ambu ResCue mask is for single use. After use, the product can be disassembled by removing the valve from the mask. The valve should be discarded after use as it cannot be cleaned. Reusing the valve may cause cross contamination. The mask can be cleaned and used again with a new valve.

#### CAUTIONS

- Don't place the used mask in the hardcase or softcase before cleaning the mask.
- After cleaning the ResCue Mask should be stored inside the case to keep clean until next use.

#### 6. Cleaning

The following hand wash cleaning procedure can be used for cleaning the mask:

- Prior to cleaning remove the valve containing the filter by twisting and pulling. Discard the valve. Remove and discard the elastic rubber band. Open the white oxygen inlet port.
- Prepare the detergent bath dissolving conventional dish-washing detergent in hot water from the tap (40-60 °C) at 1 ml per gallon (1 ml per 3.38 liters).
- Fully immerse the mask in the detergent bath placing a weight on the device to keep it submerged in the solution for 10 minutes. Agitate the mask while it is fully immersed to allow complete penetration of detergent to all parts of the mask.
- Thoroughly clean the mask with a soft bristle brush until all visible dirt is removed, paying close attention to the inside of the mask and the internal union between the cuff and the mask.
- Using a sterile syringe flush the detergent solution through the oxygen inlet lumen. A pipe cleaner or similar may be used in hard to reach areas.
- Repeat steps 2 to 5.
- Rinse the mask for one minute with hot water from the tap (40-60 °C) ensuring water is flushed through the oxygen inlet port.
- Let the device air-dry at room temperature for 10 minutes.

- After cleaning the ResCue mask should be visually inspected. If any damages are seen the ResCue Mask should be discarded.

Clean the internal side of the case with alcohol 95% and let dry before placing the clean mask back in the case.

Used products must be disposed of according to local procedures.

### **CAUTIONS**

- Do not boil, steam or gas autoclave this device!

### **NOTE**

It is the user's responsibility to evaluate the effectiveness of any deviation from the instructions provided.

## **7. Technical specifications**

Ambu ResCue Mask is in conformity with Council Directive 93/42/EEC concerning Medical Devices.

Connector Dimensions:

- Mask: Able to be connected to a standard 22 mm male connector  
Valve: Able to be connected to a standard 15 mm male connector

Dimensions: 138 mm x 113 mm x 50 mm

Weight: Approximately 117 g

## **8. Material**

- Mask: Polyvinylchloride  
Valve: Styrene-Butadiene Copolymers, Polypropylene and silicone  
Hard case: Polypropylene  
Soft case: Polyurethane, Ethylene Vinyl Acetate and Micro Fabric  
Bag: Nylon

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