**INSTRUCTIONS FOR USE**

EN

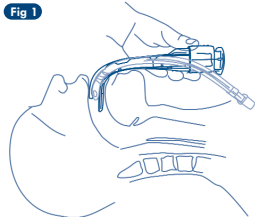
Fig 2

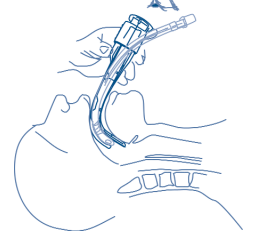
Fig 3

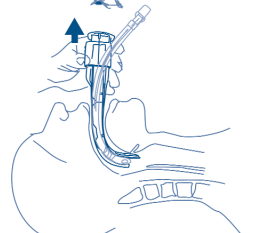
Fig 4


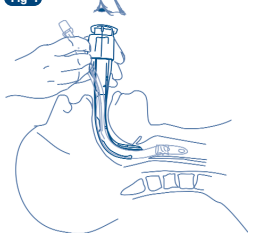
Fig 5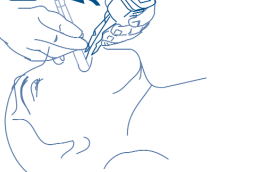

**DESCRIPTION AND INDICATIONS**

The AIRTRAQ is a SINGLE USE laryngoscope designed to facilitate intubation. It allows full visualization of the airway, during 100% of the intubation. It does not require hyperextension of the neck and permits intubating patients in virtually any position. It is provided clean, ready to use.

**SIZES**

Choose the proper size AIRTRAQ for the ETT size being used.

**Regular: ATQ-011**

Blue. For use with ETT 7.0 – 8.5  
Minimum patient mouth opening: 18 mm

**Small: ATQ-021**

Green. For use with ETT 6.0 – 7.5  
Minimum patient mouth opening: 16 mm

**Pediatric: ATQ-031**

Purple. For use with ETT 4.0 – 5.5  
Minimum patient mouth opening: 12.5 mm

**Infant: ATQ-041**

Gray. For use with ETT 2.5 – 3.5  
Minimum patient mouth opening: 12.5 mm

**TECHNIQUE FOR USING THE AIRTRAQ**

**Orotracheal Intubation**

**I. Preparation**

- Select the appropriate size AIRTRAQ based on the size ETT to be used.
- Turn ON the light using switch located on the left side of the eyepiece. Unit will flicker for 30-60 seconds until anti-fogging feature is fully activated. If no light is seen, replace unit.
- Fully deflate the ETT cuff.
- Lubricate the ETT and place it into the lateral channel of the AIRTRAQ.
- Align the tip of the ETT with the end of the lateral channel.
- Appropriately lubricate the AIRTRAQ without contacting the lens.

**II. AIRTRAQ Placement**

- Insert the AIRTRAQ into the midline of the patient's mouth. Take special care to avoid pushing the tongue toward the larynx (Fig 1).
- In some circumstances it may help to introduce the AIRTRAQ using the same technique as with a Guedel airway. However, always rotate the AIRTRAQ clockwise from the clinician's perspective.
- Slide the AIRTRAQ through the oropharynx keeping it in the midline.
- Before the AIRTRAQ reaches the vertical plane, begin looking through the eyepiece to identify airway structures (Fig 2).
- Continue insertion until the epiglottis is identified. Place the tip of the AIRTRAQ in the vallecula.
- Gently lift up the AIRTRAQ to expose the vocal cords (Fig 3). Alternatively, the tip can be placed under the epiglottis, lifting it out of the way.

**III. ETT Insertion**

- Align the vocal cords in the center of the visual field by gently moving the tip of the AIRTRAQ as needed.
  - Gently advance the ETT in the lateral channel until it is visualized passing through the vocal cords. Check insertion depth (Fig 4).
  - Inflate the ETT cuff as normal and check for proper positioning. Reposition, if needed.
- IV. AIRTRAQ Removal**
- Separate the ETT from the AIRTRAQ by pulling it laterally from the ETT, while holding the ETT in position (Fig 5).

**WARNINGS AND PRECAUTIONS**

- If utilizing any technique that requires rotation of the AIRTRAQ, always rotate clockwise from the clinician's perspective.
- This product should only be used by personnel trained in insertion of endotracheal tubes.
- Do not sterilize.
- Do not incinerate unless batteries have been removed.
- Do not submerge in liquids.
- Use only with non-flammable anesthetics.
- Do not put pressure on the teeth with this device.
- Do not force the AIRTRAQ into the upper airway.

**USAGE TIPS**

1. Initial experience should be gained in non-difficult airways.
2. Insert the AIRTRAQ carefully along the tongue, without pushing the tongue inward.
3. Keep the AIRTRAQ in the mouth's midline.
4. Look through the eyepiece before the AIRTRAQ reaches the vertical plane.
5. If structures (arytenoids, epiglottis, etc.) are not clearly recognized, withdraw the AIRTRAQ slightly.
6. Once the tip is located at the epiglottis, either at the vallecula (Macintosh style) or under the epiglottis (Miller style), gently lift up the AIRTRAQ (do not tilt or use a lever action).
7. Advance the ETT slowly without twisting it.

**BATTERY REMOVAL AND DISPOSAL (if required by facility protocol)**

- Separate the eyepiece from the main body by pulling it up. Remove the battery cover by pulling it away from the main body (pull away from the small notches near the switch).
- Remove the batteries.
- Battery shelf life is limited to expiration date; dispose of them according to established recycling policies.
- Discard the AIRTRAQ as any other potentially contaminated waste.

**WIRELESS/WIRED VIDEO SYSTEM ACCESSORY**

The manufacturer offers a Clip-On Wireless/Wired Video System to allow viewing the intubation procedure on an

external monitor. The Clip-On Video System is reusable and it is designed to be easily attached to the AIRTRAQ unit. See Instructions for Use for the Clip-On Video System.

**SINGLE-PATIENT USE**

The AIRTRAQ will function for 40 cumulative minutes of use, regardless of battery life. At 35 minutes, the AIRTRAQ will blink every 20 seconds until it reaches 40 minutes at which time the light will permanently turn OFF. The unit's performance may diminish at this point.

If at any time during extended use the unit begins to blink repeatedly (every 20 seconds), the AIRTRAQ may be reaching a low battery condition. The unit's performance may diminish at this point. **DO NOT REPLACE BATTERIES.**

This device has not been designed to be cleaned or sterilized. Use beyond this recommendation may generate serious consequences in the product's performance and will void the AIRTRAQ's warranty.

**STORAGE AND BATTERY SHELF LIFE**

The AIRTRAQ should be stored at standard room temperature (0°C to 40°C) avoiding excessively humid atmospheres. Battery shelf life is limited to expiration date.

**MANUFACTURER'S WARRANTY**

King Systems warrants the AIRTRAQ against faulty materials or manufacturing defects for only one use or until the expiration date, whichever comes first, provided that the AIRTRAQ is used in accordance with the procedures set forth in these instructions. This Warranty is applicable only if the device is purchased from an authorized distributor.

The manufacturer disclaims all other warranties, whether expressed or implied, including, without limitation, the warranties of merchantability or fitness for a particular purpose.

DEMO UNIT: Clearly labeled "DEMONSTRATION ONLY NOT FOR CLINICAL USE" demo units have the anti-fog mechanism disabled.

Manufactured in China for  
**King Systems**

© 2009 King Systems  
15011 Herriman Blvd., Noblesville, IN 46060  
317-776-6823 | www.kingsystems.com

Caution: Federal law restricts this device to the sale by or on the order of a physician.  
US Patent No. 6,843,769. AIRTRAQ is the registered trademark of Prodal Medtec S.A.

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