(English)

REF#

KAB-001

Single Use - Disposable CO₂ Absorber - Latex Free

INTENDED USE:

The KAB CO₂ absorber is intended for use during general anesthesia or patient transport whenever the clinician desires to have a single patient use, self-contained breathing system versus the conventional semi-permanent CO₂ absorbers on the anesthesia gas machines. The KAB CO₂ absorber may be used with an oxygen source during transport of the patient from surgery to recovery.

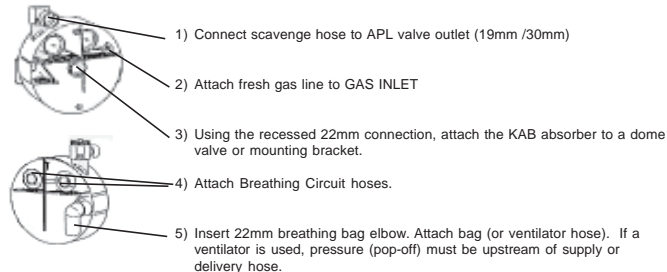
DESCRIPTION:

Disposable CO₂ absorber with directional valves and integral Adjustable Pressure Limiting (APL) valve.

Absorbent: Approximately 400 Grams (by weight)

Fittings and function meet applicable ISO and ASTM standards. Test Data available upon request.

DIRECTIONS FOR USE:

- 
- 1) Connect scavange hose to APL valve outlet (19mm /30mm)
 - 2) Attach fresh gas line to GAS INLET
 - 3) Using the recessed 22mm connection, attach the KAB absorber to a dome valve or mounting bracket.
 - 4) Attach Breathing Circuit hoses.
 - 5) Insert 22mm breathing bag elbow. Attach bag (or ventilator hose). If a ventilator is used, pressure (pop-off) must be upstream of supply or delivery hose.
- 6) TEST unit for air leakage and proper valve function before use. Replace unit immediately if there is any evidence of contamination, leakage, occlusion, or other malfunction. Ensure that all connections are secure at all times.
 - 7) Adjust APL valve for proper scavenging of excess gases. See warning statement below. NOTE: Unit is packaged with the APL valve in the OPEN position. Turn clock-wise to increase internal pressure and restrict scavenge gas flow.
 - 8) Discard when absorbent color turns to violet and according to hospital protocol.

WARNING:

- **DO NOT** connect directly to vacuum source. Only connect the scavenge hose through a waste gas scavenger interface system that has excessive positive and negative pressure relief valves.
- **DO NOT** use in presence of explosive gases or gases that are unstable to heat and/or alkali.

CAUTION:

- Federal (USA) law restricts this device to sale by or on order of a physician.
- CO₂ monitoring is recommended when using this product.

Manufacturer

KING SYSTEMS CORPORATION

Noblesville, IN USA 46060 TEL 317-776-6823 FAX 317-776-6827

EU Regulatory Representative

Medical Product Service GmbH
Borgasse 20
35619 Braunfels, Germany

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